



Easy Slides to meet all positioning needs.

- G5120S – Easy Slide Gloves
- S4040T – Easy Slide Tube 40 x 40cm
- S8080T – Easy Slide Tube 80 x 80cm
- S80120T – Easy Slide Tube 80 x 120cm
- S140100F – Easy Slide Sheet 140 x 100cm
- S140200F – Easy Slide Sheet 140 x 200cm
- S90200HO – Easy Slide Sheet 90 x 200cm with handles
- PS13746O – Positioning Slide 137 x 46cm with handles & soft fill



Easy Slide Gloves

There are hundreds of uses for Easy Slide Gloves reducing handling associated with normal slidesheets. One carer can move someone across a bed in three easy movements – 1) Slide ankles across, 2) Slide buttocks across, 3) With patient's head on a pillow, slide shoulders across. Two carers can easily slide someone up a bed, each with a glove on one hand. Place a glove under heels when sliding someone to reduce shear.



Easy Slide Tubes

Continuous tubes (40 x 40cm) to assist someone to transfer using transfer boards (shown left); or for assisting carers to slide someone across or up a bed (80 x 80cm or 80 x 120 cm) (shown right).



Easy Slide Flat Slide Sheets

Easy Slide sheets in 140 x 100cm or 140 x 200cm. Use two sheets together, one on top of the other, sliding surface inward. Carers slide the top sheet in any direction. Or fold one sheet over to slide someone across a bed. With the carer's hands on the patient's hips and shoulders gently push them away from the folded side toward the open side. Usually another carer on the open side will pull the top layer toward themselves to assist. To remove sheet after transfer pull the bottom layer towards carer.



Easy Slide Flat Slide Sheets with handles

Continuous handles around 3 sides of the slide-sheet 90 x 200cm makes it easy to grip. Use in pairs with the sliding surface face inward or use in conjunction with a normal Easy Slide flat slide-sheet 140 x 200cm.



Positioning Slide

Easily slide someone into the back of a chair. (See left). Fold the Positioning Slide with the majority out the back. Patient leaning slightly forward, their feet on the ground or on footplates. With carer's foot in the handle press down. Pull lower layer to remove. To transfer someone across surfaces, use a transfer board between a folded Positioning Slide to ease it under patient. Remove board and place under patient. Lean patient toward the direction of travel and use the handles to pull or turn the patient. (See right).



Wash at a maximum of 60°C
Do not use fabric softener or chlorine

Available from

